

SMALL IN SIZE. BIG ON ADVANTAGES.

40% HIGHER DECOMPRESSION RATE COMPARED TO AMT.⁴


MIC-KEY* GJ (14 Fr)
Feeding Tube

vs.

AMT G-JET® (14 Fr)
Feeding Tube

NEW


TUNGSTEN WEIGHTED¹

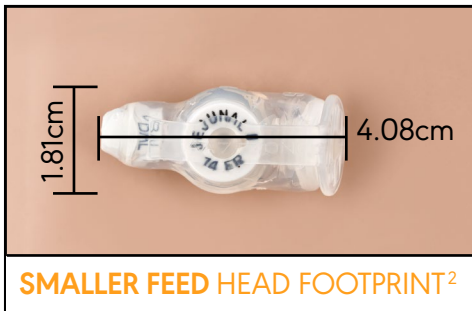

ONLY 1 TYPE OF EXTENSION SET
REQUIRED³

36% HIGHER FLOW RATE IN GASTRIC LUMEN
WHILE MAINTAINING EQUIVALENT
JEJUNAL FLOW RATE.⁷

NON-WEIGHTED

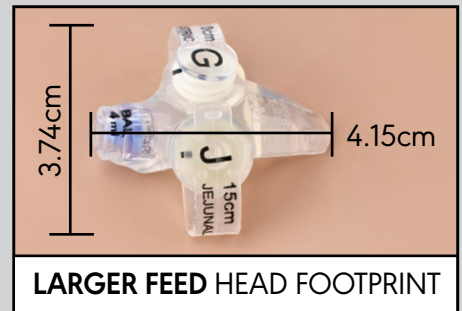

MUST HAVE 2 DIFFERENT,
PROPRIETARY EXTENSION SETS

MIC-KEY* 14 FR

GJ Feeding Tube



THE COMPLETE PORTFOLIO

From your smallest patient to your largest patient, *Avanos has you covered.*

We have the MOST comprehensive low-profile feeding tube portfolio on the market to meet your clinical needs⁵.

MIC-KEY* GJ FEEDING TUBES:

- Separate jejunal and gastric ports for simultaneous feeding and gastric decompression⁶
- Unobtrusive designs for patient comfort, concealment and security
- Wide range of tubes for the right fit, now in a 14 Fr size

		Jejunal Length (cm)			
		15	22	30	45
Stoma Length (cm)	1.0	14 Fr 16 Fr	14 Fr 16 Fr	N/A	N/A
	1.2	14 Fr 16 Fr	14 Fr 16 Fr 18 Fr	14 Fr 16 Fr 18 Fr	N/A
	1.5	14 Fr 16 Fr	14 Fr 16 Fr 18 Fr	14 Fr 16 Fr 18 Fr	14 Fr 16 Fr 18 Fr 22 Fr
	1.7	14 Fr 16 Fr	14 Fr 16 Fr 18 Fr	14 Fr 16 Fr 18 Fr	14 Fr 16 Fr 18 Fr 22 Fr
	2.0	N/A	14 Fr 16 Fr 18 Fr	14 Fr 16 Fr 18 Fr	14 Fr 16 Fr 18 Fr 22 Fr
	2.3	N/A	N/A	14 Fr 16 Fr 18 Fr	14 Fr 16 Fr 18 Fr 22 Fr
	2.5	N/A	N/A	14 Fr 16 Fr 18 Fr	14 Fr 16 Fr 18 Fr 22 Fr
	2.7	N/A	N/A	14 Fr 16 Fr 18 Fr	14 Fr 16 Fr 18 Fr 22 Fr
	3.0	N/A	N/A	N/A	14 Fr 16 Fr 18 Fr 22 Fr
	3.5	N/A	N/A	N/A	14 Fr 16 Fr 18 Fr 22 Fr

How To Configure Your MIC-KEY* GJ Feeding Tube Kit Product Code:

PLEASE NOTE: Codes beginning in 0270 are Non-ENFit[®] codes and codes beginning in 8270 are ENFit[®] codes.

NON-ENFit[®] CODE EXAMPLE: 0270 + _____ (Fr) + _____ (cm) + _____ (cm) = Product Code: _____
French Size (Outer Diameter) Stoma Length Jejunal Tail Length *NON-ENFit[®] CODE Example: 0270-14-1.0-15*

ENFit[®] CODE EXAMPLE: 8270 + _____ (Fr) + _____ (cm) + _____ (cm) = Product Code: _____
French Size (Outer Diameter) Stoma Length Jejunal Tail Length *ENFit[®] CODE Example: 8270-14-1.0-15*

1. Data on file (CLAIM 07354). 2. Data on file (CLAIM 07355). 3. Data on file (CLAIM 07356). 4. Data on file (CLAIM-07623).
 5. Data on file (CLAIM 07644). 6. Data on file (CLAIM 07429). 7. Data on file (CLAIM 07357).

For more information please visit: avanosmedicaldevices.com Call 1-844-4AVANOS (1-844-428-2667) in the United States and Canada.

AVANOS

avanosmedicaldevices.com



*Registered Trademark or Trademark of Avanos Medical, Inc., or its affiliates.

© 2019 AVNS. All rights reserved.

® ENFit is a Registered Trademark or Trademark of Global Enteral Device Supplier Association, Inc.

Used with permission.

© AMT G-JET is a trademark of Applied Medical Technology, Inc.

COPY-03206