

This checklist was designed to help you inform your patients of what they need to know to help ensure successful enteral feeding once they leave the medical setting.



MIC-KEY* GJ Quick Start Guide, Feeding Tube Equipment & MIC-KEY* Feeding Tube Demo Doll

- Review the Quick Start Guide with the patient (and/or caregiver) along with each piece of equipment in their kit, including the syringes and extension sets. Explain proper use and care and discuss how often each item should be replaced.
- Inform the patient and/or caregiver of how/when/why they will be receiving replacement parts and what to do with them.
- Have the patient and/or caregiver practice setting up their first feeding by using their own equipment on the MIC-KEY* Feeding Tube Demo Doll.
- Go over the **"Tips for Feeding Tube Success"** section of the Quick Start Guide in detail to assure that the patient and/or caregiver fully understands the do's and don'ts of MIC-KEY* GJ equipment use.
- Point out the information resources that are available in the **"For More Information"** section of the Quick Start Guide and discuss their purposes.



Stoma Site Care & Signs To Watch For

- Describe to the patient and/or caregiver how to care for their stoma site.
- Refer to the Stoma Site Poster when discussing how to recognize stoma site issues, how to prevent them and what to do if issues arise.
- Explain why different patients need different feeding tube lengths.
- Inform the patient and/or caregiver that they may need to be resized for feeding tube length if:
 - They experience any changes in weight.
 - They notice any stoma issues listed in the Quick Start Guide or Stoma Site Poster.

ENTERAL FEEDING Stoma Site and Enteral Feeding Tube			
STOMA SITE COMPLICATIONS			
Complication	Possible Cause	Characteristics	Management - Feeding Tube
Stoma Bleeding	Stoma irritation, trauma, or infection	Stoma is bright red, swollen, and oozing blood	Stop feeding immediately. Notify the physician. Do not attempt to flush the tube.
Stoma Pain	Stoma irritation, trauma, or infection	Stoma is red, swollen, and painful	Stop feeding immediately. Notify the physician. Do not attempt to flush the tube.
Stoma Discharge	Stoma irritation, trauma, or infection	Stoma is red, swollen, and has a discharge	Stop feeding immediately. Notify the physician. Do not attempt to flush the tube.
Stoma Necrosis	Stoma irritation, trauma, or infection	Stoma is dark red, black, and necrotic	Stop feeding immediately. Notify the physician. Do not attempt to flush the tube.
Stoma Stenosis	Stoma irritation, trauma, or infection	Stoma is narrow and difficult to pass a tube	Stop feeding immediately. Notify the physician. Do not attempt to flush the tube.
Stoma Prolapse	Stoma irritation, trauma, or infection	Stoma is protruding and difficult to pass a tube	Stop feeding immediately. Notify the physician. Do not attempt to flush the tube.
Stoma Infection	Stoma irritation, trauma, or infection	Stoma is red, swollen, and has a discharge	Stop feeding immediately. Notify the physician. Do not attempt to flush the tube.
Stoma Ulcer	Stoma irritation, trauma, or infection	Stoma is red, swollen, and has a sore	Stop feeding immediately. Notify the physician. Do not attempt to flush the tube.
Stoma Perforation	Stoma irritation, trauma, or infection	Stoma is red, swollen, and has a hole	Stop feeding immediately. Notify the physician. Do not attempt to flush the tube.
Stoma Stricture	Stoma irritation, trauma, or infection	Stoma is narrow and difficult to pass a tube	Stop feeding immediately. Notify the physician. Do not attempt to flush the tube.
Stoma Stenosis	Stoma irritation, trauma, or infection	Stoma is narrow and difficult to pass a tube	Stop feeding immediately. Notify the physician. Do not attempt to flush the tube.
Stoma Prolapse	Stoma irritation, trauma, or infection	Stoma is protruding and difficult to pass a tube	Stop feeding immediately. Notify the physician. Do not attempt to flush the tube.
Stoma Infection	Stoma irritation, trauma, or infection	Stoma is red, swollen, and has a discharge	Stop feeding immediately. Notify the physician. Do not attempt to flush the tube.
Stoma Ulcer	Stoma irritation, trauma, or infection	Stoma is red, swollen, and has a sore	Stop feeding immediately. Notify the physician. Do not attempt to flush the tube.
Stoma Perforation	Stoma irritation, trauma, or infection	Stoma is red, swollen, and has a hole	Stop feeding immediately. Notify the physician. Do not attempt to flush the tube.
Stoma Stricture	Stoma irritation, trauma, or infection	Stoma is narrow and difficult to pass a tube	Stop feeding immediately. Notify the physician. Do not attempt to flush the tube.


Formula, Hydration & Medications

- Refer to the physician's diet instructions and discuss formula and blenderized diets.
- Review their physician's instructions for hydration and fluid intake.
- Explain how to administer medications properly using a feeding tube and extension set and review the instructions for medications.

Contact Information & Gastro-Jejunal Info Card

- Provide the patient and/or caregiver with your contact information.
- Also, provide them with the contact information of their clinical team.
- Explain who to contact, when and for what reasons.
- Log this contact information on the patient's Gastro-Jejunal Info Card for easy reference.

MIC-KEY* GJ		AVANOS
Your Gastric-Jejunal Feeding Tube Information		
Name _____	Stoma length in centimeters _____	Amount of each feeding _____
Phone _____	Ballon volume _____	Amount of water _____
Physician _____	Lot number _____	Pump setting or flow rate _____
Phone _____	MIC-KEY* continuous feed extension set (if per case), change every 2 weeks.	Additional ingredients: _____
Other medical professional _____	Order# _____	_____
Phone _____	MIC-KEY* bolus feed extension set (if per case), change every 2 weeks.	_____
Placement date _____	Order# _____	Blenderized table food: follow the doctor's instructions. Flush with _____ ml water before and after every feeding.
Tube type _____	_____	_____
Tube replacement dates _____	FORMULA PREPARATION	_____
_____	Type of formula _____	_____
TUBE SPECIFICATIONS	Amount of formula _____	_____
French size _____	Feeding time _____	_____
Length in centimeters _____	_____	Follow your medical professional's instructions.
avanosmedicaldevices.com		*Registered Trademark or Trademark of Avanos Medical, Inc., or its affiliates. © 2018 AVNS. All rights reserved. C172061 COPY-01928

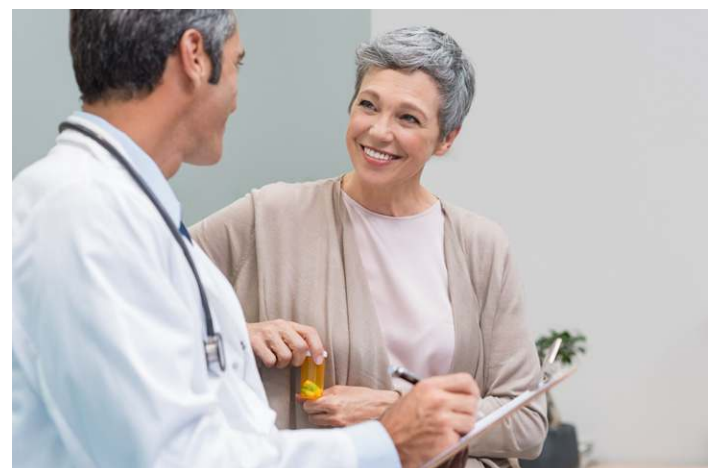

Patient Resources
www.mic-key.com
www.mic-key.com/newsletter
www.facebook.com/mymickeytube