

NEOMED* ENFit® CLEANING TOOL




The NEOMED* ENFit® cleaning tool is used to remove debris from a NEOMED* feeding tube or extension set with an ENFit male connector.

1

Always use aseptic technique. Remove closure cap from the hub.

2

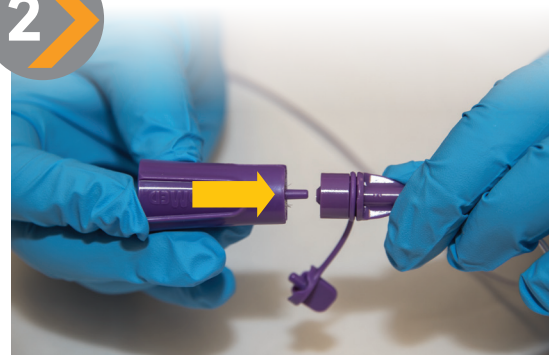

Insert cleaning tool plug into the hub. Do not overinsert the plug, as this may lead to difficulty removing the cleaning tool from the hub.

3



Slide the cleaning tool over the hub to engage internal bristles.

4



Rotate the cleaning tool to dislodge debris.

5



Rotate the cleaning tool clock-wise at least 360° to sweep debris to the bottom of the hub. The Cleaning Tool features openings that allow for the use of cleaning agents (distilled water, saline, etc.)

6



Align the cleaning tool and gently push bristles through bottom of hub to eject debris.

7

Remove expelled debris per hospital protocol.

8



Carefully remove the cleaning tool and dispose as required by hospital protocol and local or national regulations.

WARNING

CAUTION

DO NOT autoclave or re-use the NEOMED* ENFit® cleaning tool as this may increase the risk of biological infection. Ensure the plug stays engaged during use.

