

ENFIT® TAMPER-EVIDENT CAP INSTRUCTIONS

OPERATING INSTRUCTIONS



Insert ENFit® Syringe into Tamper Evident Cap and rotate clockwise while pushing downwards **(into cap)**.




Verify installation by rotating the cap until snug. This typically requires less than 1/4 turn.

TO ACCESS & USE MEDICATION



Visually inspect Tamper Evident Cap for evidence that the syringe has been previously used or tampered with, such as:

- Missing components **(outer cover A)**
- Visible damage

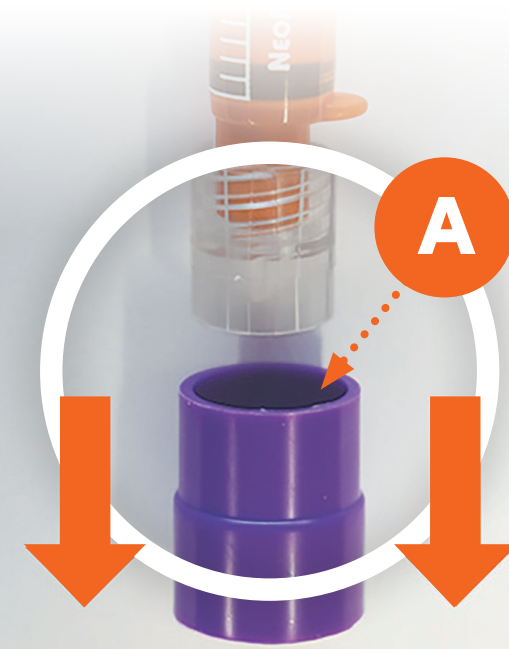

Remove the Tamper Evident Cap's outer cover **(A)** by pulling it directly away from the attached syringe.



Remove tip cap **(B)** from syringe by twisting it counter-clockwise.

CAUTION

- Damaged or missing Tamper Evident outer cover (A) indicated potential tampering and syringe should be discarded.
- This is a single use device. Do not re-use or re-attach the tip cap (B) after it has been removed from the syringe.
- Properly installed cap remains sealed on syringe until removed.
- Do not use if package is opened or damaged.