

# ENTERAL FEEDING WITH ENFit®

## FILLING



### Straw

1. Twist the straw onto the syringe. Place the straw into the bottle or other container.
2. To help prevent air bubbles, gently but quickly pull and push the syringe plunger back and forth to cycle it one or more times.
3. Carefully remove any air pockets. Verify dose then disconnect straw.

### Straw Priming Volumes

- 3" (8 cm) Small Bore: 0.1 mL
- 3" (8 cm) Large Bore: 0.4 mL
- 6" (15 cm) Large Bore: 0.7 mL



### Transfer Lid

1. Select the breastmilk or formula container and attach transfer lid.
2. To disconnect syringe, return bottle to upright position, then carefully remove any air pockets as appropriate.
3. Remove the syringe then close the tethered plug.

### CAUTION

If fluid needs to be added into the bottle, it must be in an **upright position** or leakage will occur (syringe tip must be down).



### Bottle


1. Fill the syringe to desired volume.
2. When filling the low dose tip syringe, if there is fluid in moat (the area between the male lumen and the outer ring that lies outside of the fluid path), tap to remove.

### CAUTION

Remove fluid from moat before delivery.




## TRANSPORT

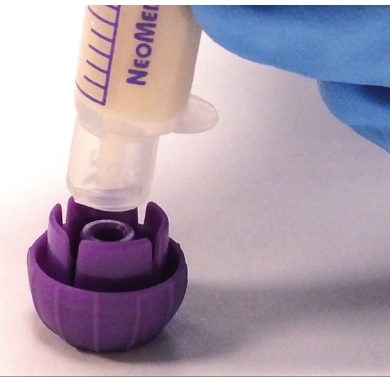

### Transport

1. Place the Self-Righting Tip Cap on a flat surface or hold firmly in gloved hand.
2. Push the cap onto the syringe, or twist on by hand.

## POST FEED

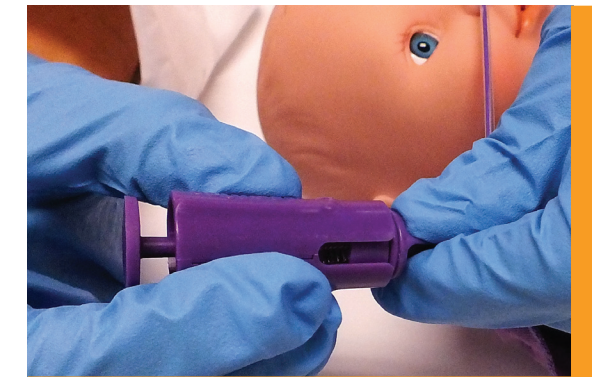

1. Upon completion of feed, rotate hub counterclockwise to disconnect syringe/extension set from the feeding tube.
2. Inspect the feeding tube hub for moisture or residual material.
3. Clean the hub using the NeoMed cleaning tool in accordance with department protocol.
4. Immediately after cleaning the feeding tube hub, plug the feeding tube opening with the tethered cap.